

Did you know that ...

EVERYTHING DEPENDS ON EVERYTHING ELSE !

Today more than before time is money - we have to plan our schedules even more efficiently. A well-organised, properly planned day makes it easier to do our job effectively and successfully. Satisfaction with our performance at work is a good way to begin our leisure time. What can the LM-Servo instrument cassette contribute to the modern dentist's office ?

LM-SERVO IMPROVES ASEPSIS

The less frequently instruments are handled the longer they remain sterile. By using the LM-Servo the necessity of replacing the instruments in the correct order after every use is avoided. The instruments are arranged in the cassette only once, in other words when a particular set of instruments is first removed for use. The same instruments are kept in the same cassette on the instrument tray and throughout the whole maintenance cycle. After the dentist or dental hygienist has finished a patient's treatment, the instruments are securely locked in place in the cassette until they are required again or are removed for sharpening. The LM-Servo can be safely placed in disinfectant, an ultrasonic cleaner or an autoclave. It is also easy to pack the cassettes in sterilisation bags. The instruments are protected and kept in the correct order during the entire maintenance cycle.

SAVES TIME

The LM-Servo enables the dentist or dental hygienist to use their valuable time in patient care instead of dealing with the instruments. The time required for maintenance decreases by some 20 %, as the instruments are arranged only once.

PROTECTS YOUR INSTRUMENTS

The cassette prevents the instruments from coming into contact with each other and keeps them cleaner longer. Blunting and other damage is also reduced. Practical experience has shown that instrument costs decrease by about 15 %.

IMPROVES ORDER

Thanks to the colour-coding, cassettes and instruments are always returned to their right owner. This means that dental



hygienists no longer have to search drawers and cupboards for misplaced instruments. The cassettes can be marked with code buttons, for example, according to treatment procedure, enabling the required cassette to be found quickly and easily. If several dentists work in the same room, the

cassettes can be marked according to room and individual dentist.

SECURITY FOR DENTAL HYGIENISTS

Seeing the dentist or dental hygienist remove an LM-Servo from its sterilisation bag reassures the patient that the instruments are absolutely sterile - making him feel that he is in safe hands.

A QUIETER ENVIRONMENT

There is often a lot of noise in the dental professional's working environment, for example, from ultrasonic apparatus, drills and suction units. The cassettes, which are made from PPS plastic, do not increase noise in the treatment or maintenance room. A harmonious working community reduces stress and improves concentration. This in turn produces results that satisfy both you and your patient.

In short: **EVERYTHING AFFECTS EVERYTHING ELSE !**

Turku, February 1998

Tina Norrman
Dental Hygienist

