

Sharpening machine

LM-RondoPlus sharpening machine

Instruments used in treating hard tissue blunt quickly, naturally compromising efficiency and results. Periodontal instruments should be sharpened regularly.

Easy and quick

- simple method - the blade is held in the correct sharpening position and the grinding stone rotates
- sharpening one instrument only takes a few seconds

Versatile

- for both periodontal and preparation instruments
- for both right or left-handed
- the direction of rotation can be selected

Accurate and safe

- a dual-position instrument rest. The upper position is for preparation instruments and the lower for periodontal instruments
- the original blade form is preserved
- low rotational speed - the blade will not heat up nor will its temper be affected

The fine-grain ceramic stone is 400 grit. Finer and coarser sharpening stones are available as spare parts.

On either side of the LM-RondoPlus there are LM-Fingo test sticks. Sharpness can be tested both with Fingo mounted on the LM-RondoPlus or your finger. The Fingo is made of an acrylic plastic and is sized to fit all users.

LM-RondoPlus LM 8600


Package includes: sharpening machine, foot pedal, 400 grit sharpening stone, 2 LM-Fingos, 5 emery tapes, instruction CD and quick guide.

Spare Parts

Grinding stone	
600 grit (white)	LM 8660
400 grit, coarser (grey)	LM 8640
320 grit, coarsest (black)	LM 8632
Foot pedal	LM 8630
Emery tapes, 5 pcs	LM 8550
LM-Fingo, 2 pcs	LM 8602
LM-Fingo, 6 pcs	LM 8606



PATENTED

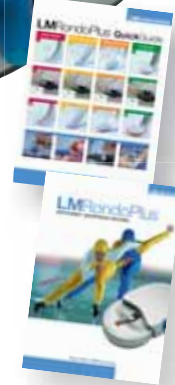

Interactive demonstration and instruction CD and handy quick guide included!

